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### Instructions for Use

#### In case of a tooth accident:

- Search for avulsed tooth or tooth fragment immediately and insert it in the solution in the **SOS Zahnbox** instantaneously.
- Do not touch the tooth at the root or area of fracture!
- Do not clean or disinfect the tooth!
- Open the **SOS Zahnbox**, insert the tooth or the tooth fragment in the solution and re-close.
- Note patient's name, date and time on the bottle label. Consult a dentist or a dental clinic as soon as possible.

#### Information for the treating dentist:

Prior to its replantation, the tooth needs to be rinsed thoroughly with the NaCl solution. The solution in the **SOS Zahnbox** keeps the cells at the tooth root viable for up to 48 hours at room temperature.

The chances for a successful replantation increase if the tooth or the tooth fragment is transferred into a new **SOS Zahnbox** after 24 hours.

#### Application:

Storing of

- avulsed teeth – which are to be replanted
- extracted teeth – which are to be replanted or transplanted
- tooth fragments – which are to be reattached

#### Observe expiry date!

The miradent **SOS Zahnbox** contains a ready-for-use solution. If this solution turns turbid and changes colour from light red to yellow, the liquid is contaminated and must no longer be used.

This product is for single use only and cannot be sterilized after opening. Re-use will have a negative impact on the asepsis, which can then no longer be guaranteed, and which will lead to a high risk of infection.

Keep out of reach of children.

Storage temperature: 4 °C to 25 °C

Not for intake. Do not open prior to use!

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