B. Braun Closure Technologies

ManiplerAZ[®]-35W

(GB)

Instructions for use

Single use skin stapler

(D)

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(F)

Mode d'emploi

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(E)

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Grapadora cutánea desechable

(I)

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Suturatrice cutanea monouso

(P)

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ManiplerAZ®-35W Single use skin stapler

Description

The ManiplerAZ® is a sterile, single use skin stapler designed to deliver 35 stainless steel staples for routine wound closure.

Indications

The ManiplerAZ® skin stapler can be used for routine skin closure in a wide variety of surgical procedures.

Application

- Prior to using the skin stapler verify compatibility of the skin staple remover.
- Using sterile technique, remove the skin stapler from the package.
- With a tissue forceps, pull skin edges together until edges evert (Fig. 1).
- Position the skin stapler over the center of the everted skin edges (Fig. 2).
- To form the staple, squeeze the trigger firmly together until the trigger motion is halted (Fig. 3).
- Release the trigger (Fig. 4).
- Move the stapler off the incision (Fig. 5).
- Always verify the correct position of the staples.

Contra-indications

- When it is not possible to maintain at least 5 mm distance from the stapled skin to underlying bones, vessels, or internal organs, the use of staples for skin closure is contraindicated.
- The ManiplerAZ® staples contain nickel, which might lead to allergic reactions with some patients.

Safety instructions

- Do not use the ManiplerAZ® if the date of expiry has passed.
- Store the ManiplerAZ® at room temperature and use it directly after the package has been opened.
- Never use products from open or damaged sterile packaging.
- Do not resterilize, as this might impair the functioning of the product and injure the patient.
- Use the ManiplerAZ® skin stapler only on one patient.
- After using, dispose of the product properly as medical waste.

Sterility

Sterilization is effective until the date of expiry, if the package is stored correctly in a closed and intact package.

Shelf life is 5 years.

Delivery formats und package sizes

The ManiplerAZ®-35W is delivered in a box of 6 units.

Explanation of the symbols on the packaging

	Do not reuse!
	Use until year + month
	Sterile, unless package is opened or damaged. Method of sterilization – Ethylene oxide
	CE marking and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC
LOT	Batch (Lot) number
	Please follow the instructions for use!
REF	Art no. 783100
DIM	Staple size 6.9 mm x 3.6 mm

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Aesculap AG